



INHEALTH[®]
T E C H N O L O G I E S

Blom-Singer[®]

voice restoration systems

*The Voice of Experience
Since 1978*

2010 Product Catalog

Introducing the New Blom-Singer® Dual Valve Voice Prosthesis




Our new dual valve Indwelling voice prosthesis uses leading-edge design to deliver on a very simple, but powerful benefit: two valves are better than one!

Should the primary valve fail due to Candida (yeast) colonization, the secondary valve is intended to remain viable to help prevent leakage of fluids.

As an added advantage, each valve is imbedded with silver oxide, an antimicrobial agent that provides significant protection to the device for laryngectomees who are susceptible to Candida growth. Blom-Singer voice prostheses that contain silver oxide have been proven to eliminate or reduce the need for antifungal agents, such as Nystatin™ or Mycelex®.

Included with the Dual Valve Voice Prosthesis is a unique pop-through insertion system, designed to assist the clinician in the deployment of the esophageal flange during placement.

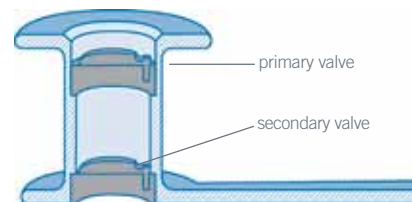

The Dual Valve voice prostheses are offered in non-sterile 20 Fr. indwelling packages, in four lengths:

| LENGTH | 20 FR DIAMETER |
|--------|----------------|
| 8mm | DV2008-NS |
| 10mm | DV2010-NS |
| 12mm | DV2012-NS |
| 14mm | DV2014-NS |

Blom-Singer®

voice restoration systems

*The Voice of Experience
Since 1978*


Table of Contents

In 1978, Drs. Singer and Blom pioneered the methods of tracheoesophageal puncture and valved silicone voice prostheses that for over the past 30 years have become the international standard for voice restoration, allowing thousands of patients worldwide to regain their ability to speak. As a leading supplier of advanced voice restoration systems, our goal at InHealth is to help laryngectomees regain and preserve a vital part of their lives – their voice.



| | | |
|----------------|--|----|
| Section One: | <i>Blom-Singer® Voice Prostheses Restoration System</i> | 2 |
| Section Two: | <i>Blom-Singer® Voice Prosthesis Accessories and Supplies</i> | 10 |
| Section Three: | <i>Skin Care & Scar Management Gel</i> | 14 |
| Section Four: | <i>Stoma Products, Personal Needs & Stoma Illumination</i> | 18 |
| Section Five: | <i>Professional Products and ENT Products</i> | 19 |
| Section Six: | <i>Educational Videos, Books and Training Aids</i> | 22 |
| Section Seven: | <i>Customer Service & Ordering Information</i> | 24 |

Blom-Singer® Voice Restoration System

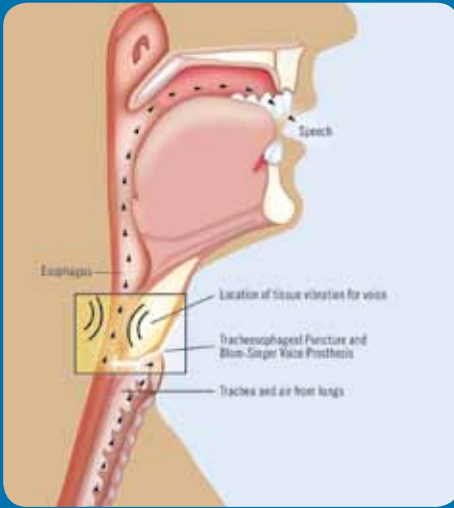

VOICE PRODUCTION

By creating a small surgical puncture just inside the stoma, from the back wall of the trachea into the esophageal wall, a voice prosthesis can be placed into this passage to enable tracheoesophageal speech. Voice is produced by temporarily blocking the stoma so that exhaled air from the lungs can be directed from the trachea through the prosthesis into the esophagus and then out through the mouth. Fluent, conversational speech may be acquired within just a few days.

Styles



Blom-Singer Duckbill




Blom-Singer Low Pressure

Patient-changeable prostheses that patients may insert, remove and clean.

Blom-Singer®

voice restoration systems

*The Voice of Experience
Since 1978*

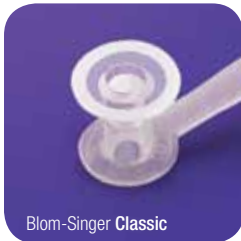

Blom-Singer Classic



Blom-Singer Advantage



Blom-Singer Dual Valve

Clinician-placed (indwelling) voice prostheses are for laryngectomees who are unable or reluctant to perform the routine removal and insertion necessary for the cleaning and maintenance of patient-changeable voice prostheses.

Rapid Response special order voice prostheses are offered with a wide range of features and sizes for patients who have special needs.

Advantage and Dual Valve voice prostheses are for patients who are susceptible to Candida growth. Both valves are imbedded with silver oxide, an antimicrobial agent that has been proven to eliminate or reduce the need for antifungal agents such as Nystatin™ or Mycelex®.

–**Color Coding** has been applied to products throughout this catalog to assist the user in identifying the compatible accessories for the largest selection of styles and sizes of voice prostheses available.

- Blom-Singer **Advantage**
- Blom-Singer **Dual Valve**
- Blom-Singer **Classic**
- Blom-Singer **Low Pressure**
- Blom-Singer **Duckbill**

Blom-Singer® Voice Prostheses—Clinician-placed



Blom-Singer® Advantage® Indwelling Voice Prosthesis

Introduced in 2003, the Advantage indwelling voice prosthesis is the first prosthesis ever to incorporate a valve imbedded with silver oxide as a material preservative**. This feature may eliminate or reduce the need for anti-fungal agents such as Nystatin for some users, but not for others.

| Length | SOFT VALVE ASSEMBLY | | | HARD VALVE ASSEMBLY | |
|--------|---------------------|-----------|----|---------------------|----|
| | 16 Fr. | 20 Fr. | Rx | 20 Fr. | Rx |
| 4mm | AS1604-NS | AS2004-NS | • | AD2004-NS* | • |
| 6mm | AS1606-NS | AS2006-NS | • | AD2006-NS* | • |
| 8mm | AS1608-NS | AS2008-NS | • | AD2008-NS* | • |
| 10mm | AS1610-NS | AS2010-NS | • | AD2010-NS* | • |
| 12mm | AS1612-NS | AS2012-NS | • | AD2012-NS* | • |
| 14mm | AS1614-NS | AS2014-NS | • | AD2014-NS* | • |
| 16mm | AS1616-NS | AS2016-NS | • | AD2016-NS* | • |
| 18mm | AS1618-NS | AS2018-NS | • | AD2018-NS* | • |
| 20mm | AS1620-NS | AS2020-NS | • | AD2020-NS* | • |
| 22mm | AS1622-NS | AS2022-NS | • | AD2022-NS* | • |

Nonsterile—Includes nonsterile voice prosthesis for secondary or replacement TEP procedures; with insertion/cleaning accessories. Clinician-placed.

*Not available in England, Germany, Sweden, or The Netherlands.



Blom-Singer® Classic™ Indwelling Voice Prosthesis

Since its introduction in 1994 as the first indwelling style voice prosthesis, the Classic indwelling voice prosthesis has been a premier voice restoration product for laryngectomees worldwide. It is designed with increased esophageal and tracheal flanges for maximum retention and incorporates a radiopaque ring for visualization under x-ray.

| Length | 16 Fr. | 20 Fr. | Rx |
|--------|-----------|-----------|----|
| 4mm | IN1604-NS | IN2004-NS | • |
| 6mm | IN1606-NS | IN2006-NS | • |
| 8mm | IN1608-NS | IN2008-NS | • |
| 10mm | IN1610-NS | IN2010-NS | • |
| 12mm | IN1612-NS | IN2012-NS | • |
| 14mm | IN1614-NS | IN2014-NS | • |
| 16mm | IN1616-NS | IN2016-NS | • |
| 18mm | IN1618-NS | IN2018-NS | • |
| 20mm | IN1620-NS | IN2020-NS | • |
| 22mm | IN1622-NS | IN2022-NS | • |

Nonsterile—Includes nonsterile voice prosthesis for secondary or replacement TEP procedures; with insertion/cleaning accessories. Clinician-placed.



Blom-Singer® Classic™ Indwelling Voice Prosthesis—Sterile Series

Available for primary TEP placement procedures.

| Length | 16 Fr. | 20 Fr. | Rx |
|--------|-----------|-----------|----|
| 8mm | IN1608-SO | IN2008-SO | • |
| 10mm | IN1610-SO | IN2010-SO | • |
| 12mm | IN1612-SO | IN2012-SO | • |
| 14mm | IN1614-SO | IN2014-SO | • |

Sterile—Includes sterile voice prosthesis for primary TEP procedures; with insertion/cleaning accessories. Clinician-placed.



Blom-Singer® Dual Valve™ Indwelling Voice Prosthesis

Introducing the first ever dual-valve prosthesis. Should the primary speech valve fail due to Candida (yeast) colonization, the secondary speech valve is intended to remain viable to help prevent leakage of fluids.

| Length | 20 Fr. | Rx |
|--------|-----------|----|
| 8mm | DV2008-NS | • |
| 10mm | DV2010-NS | • |
| 12mm | DV2012-NS | • |
| 14mm | DV2014-NS | • |

Nonsterile—Includes nonsterile voice prosthesis for secondary or replacement TEP procedures; with insertion/cleaning accessories. Clinician-placed.

NEW

LEGEND

Voice prosthesis color designations denote compatibility with other products.

 Blom-Singer Advantage

 Blom-Singer Dual Valve

 Blom-Singer Classic

 Blom-Singer Low Pressure

 Blom-Singer Duckbill

Rx

Products with a Rx notation require a USA prescription from a qualified medical professional

**Silver oxide, a known anti-fungal substance, has been incorporated into the silicone flap valve of the Blom-Singer Advantage Indwelling Voice Prosthesis as a material preservative.

Blom-Singer® Voice Prostheses—Patient-changeable



Blom-Singer Low Pressure Voice Prosthesis

The Low Pressure is a patient-changeable, one-way flapper valve prosthesis that the patient may insert, remove and clean.

16 Fr. Low Pressure

| Length | 16 Fr. | Rx |
|--------|----------|----|
| 6mm | BE6009 | • |
| 8mm | LP16-008 | • |
| 10mm | BE6010 | • |
| 12mm | LP16-012 | • |
| 14mm | BE6011 | • |
| 18mm | BE6012 | • |
| 22mm | BE6013 | • |
| 25mm | BE6014 | • |
| 28mm | BE6015 | • |

20 Fr. Low Pressure

| Length | 20 Fr. | Rx |
|--------|----------|----|
| 6mm | BE2014 | • |
| 8mm | LP20-008 | • |
| 10mm | BE2018 | • |
| 12mm | LP20-012 | • |
| 14mm | BE2022 | • |
| 18mm | BE2026 | • |
| 22mm | BE2030 | • |
| 25mm | BE2033 | • |

Special Order Blom-Singer Low Pressure Voice Prosthesis

We are dedicated to providing voice restoration products that accommodate special needs; please allow up to six weeks for delivery. *Patient-changeable*

| Length | 16 Fr. Special Increased Resistance | 16 Fr. Special Length | 16 Fr. Special Long Strap | 18 Fr. Low Pressure | 20 Fr. Special Length | 20 Fr. Special Increased Resistance | Rx |
|--------|-------------------------------------|-----------------------|---------------------------|---------------------|-----------------------|-------------------------------------|----|
| 4mm | — | 11-902-06 | — | — | 11-251-06 | — | • |
| 6mm | LP16-006-IR | — | 11-004-00 | — | — | LP20-006-IR | • |
| 8mm | LP16-008-IR | — | — | — | — | LP20-008-IR | • |
| 10mm | LP16-010-IR | — | 11-004-01 | 11-243-02 | — | LP20-010-IR | • |
| 12mm | LP16-012-IR | — | — | — | — | LP20-012-IR | • |
| 14mm | LP16-014-IR | — | 11-004-02 | — | — | LP20-014-IR | • |
| 16mm | — | 11-902-04 | — | — | 11-251-04 | — | • |
| 18mm | LP16-018-IR | — | 11-004-03 | — | — | LP20-018-IR | • |
| 20mm | — | 11-902-05 | — | — | 11-251-05 | — | • |
| 22mm | LP16-022-IR | — | 11-004-04 | — | — | LP20-022-IR | • |
| 25mm | LP16-025-IR | — | 11-004-05 | — | — | LP20-025-IR | • |
| 28mm | LP16-028-IR | — | — | — | — | — | • |



Blom-Singer Duckbill Voice Prosthesis

The Duckbill is a patient-changeable, one-way slit valve prosthesis that the patient may insert, remove and clean.

16 Fr. Duckbill

| Length | 16 Fr. | Rx |
|--------|----------|----|
| 6mm | DB16-006 | • |
| 8mm | DB16-008 | • |
| 10mm | BE6000 | • |
| 12mm | DB16-012 | • |
| 14mm | BE6001 | • |

| Length | 16 Fr. | Rx |
|--------|--------|----|
| 18mm | BE6002 | • |
| 22mm | BE6003 | • |
| 25mm | BE6004 | • |
| 28mm | BE6005 | • |

Special Order Blom-Singer Duckbill Voice Prosthesis

We are dedicated to providing voice restoration products that accommodate special needs; please allow up to six weeks for delivery. *Patient-changeable*

16 Fr. Duckbill Special Lengths

| Length | 16 Fr. | Rx |
|--------|-------------------|----|
| 16mm | SPECIAL 11-901-04 | • |
| 20mm | SPECIAL 11-901-05 | • |

16 Fr. Duckbill Long Strap

| Length | 16 Fr. | Rx |
|--------|-------------------|----|
| 10mm | SPECIAL 11-003-01 | • |
| 14mm | SPECIAL 11-003-02 | • |
| 18mm | SPECIAL 11-003-03 | • |
| 22mm | SPECIAL 11-003-04 | • |
| 25mm | SPECIAL 11-003-05 | • |
| 28mm | SPECIAL 11-003-06 | • |

LEGEND

Voice prosthesis color designations denote compatibility with other products.

-  Blom-Singer Advantage
-  Blom-Singer Dual Valve
-  Blom-Singer Classic
-  Blom-Singer Low Pressure
-  Blom-Singer Duckbill

Rx

Products with a Rx notation require a USA prescription from a qualified medical professional

Blom-Singer® Rapid Response Voice Prosthesis

■ Special Order Voice Prosthesis Program

People are different, and a standard voice prosthesis may not be right for everyone. That's why we've introduced the new **Rapid Response Special Order Voice Prosthesis Program**. Now you can order from a wide range of specially designed products.



Now Available!

Special Length:

Available in one millimeter incremental lengths for comfort and a more precise TE puncture fit.

Large Esophageal Flange:

Provides a "seal" to help manage leakage around a voice prosthesis. Also, enables a safe and convenient downsizing from a wider diameter prosthesis to a smaller diameter.

Effectiveness is dependent on an optimal prosthesis length fit that achieves a snug flange-to-flange seal against the wall surface(s).

Large Esophageal & Tracheal Flanges:

Provides a "double seal" in patients with significantly enlarged puncture tracts and "tear-drop" or irregularly shaped punctures.

Effectiveness is dependent on an optimal prosthesis length fit that achieves a snug flange-to-flange seal against the wall surface(s).

Increased Resistance:

A voice prosthesis with a valve that opens less easily. Designed to be used in patients in whom conventional voice prostheses open intermittently with fluctuations in esophageal pressures resulting in air ingestion and/or leakage.

TEP Occluder:

Voice Prosthesis with its valve permanently sealed closed.

■ Special Length



Special Length

| | 5mm | 7mm | 9mm | 11mm | 13mm | 25mm | Rx |
|--------|-----------|-----------|-----------|-----------|-----------|-----------|----|
| 16 Fr. | IN1605-SL | IN1607-SL | IN1609-SL | IN1611-SL | IN1613-SL | — | • |
| 20 Fr. | IN2005-SL | IN2007-SL | IN2009-SL | IN2011-SL | IN2013-SL | IN2025-SL | • |



Large Esophageal Flange / Special Length

| | 5mm | 7mm | 9mm | 11mm | 13mm | 25mm | Rx |
|--------|-------------|-------------|-------------|-------------|-------------|------|----|
| 16 Fr. | IN1605-LESL | IN1607-LESL | IN1609-LESL | IN1611-LESL | IN1613-LESL | — | • |
| 20 Fr. | IN2005-LESL | IN2007-LESL | IN2009-LESL | IN2011-LESL | IN2013-LESL | — | • |



Large Esophageal Flange / Special Length / Increased Resistance

| | 5mm | 7mm | 9mm | 11mm | 13mm | 25mm | Rx |
|--------|-------------|-------------|-------------|-------------|-------------|------|----|
| 16 Fr. | IN1605-LESI | IN1607-LESI | IN1609-LESI | IN1611-LESI | IN1613-LESI | — | • |
| 20 Fr. | IN2005-LESI | IN2007-LESI | IN2009-LESI | IN2011-LESI | IN2013-LESI | — | • |

■ Large Esophageal Flange



Large Esophageal Flange

| | 4mm | 6mm | 8mm | 10mm | 12mm | 14mm | Rx |
|--------|------------|------------|------------|------------|------------|------------|----|
| 16 Fr. | IN1604-LEF | IN1606-LEF | IN1608-LEF | IN1610-LEF | IN1612-LEF | IN1614-LEF | • |
| 20 Fr. | IN2004-LEF | IN2006-LEF | IN2008-LEF | IN2010-LEF | IN2012-LEF | IN2014-LEF | • |



Large Esophageal Flange (Esophageal & Tracheal)

| | 4mm | 6mm | 8mm | 10mm | 12mm | 14mm | Rx |
|--------|-----------|-----------|-----------|-----------|-----------|-----------|----|
| 16 Fr. | IN1604-LF | IN1606-LF | IN1608-LF | IN1610-LF | IN1612-LF | IN1614-LF | • |
| 20 Fr. | IN2004-LF | IN2006-LF | IN2008-LF | IN2010-LF | IN2012-LF | IN2014-LF | • |



Large Esophageal Flange / Increased Resistance

| | 4mm | 6mm | 8mm | 10mm | 12mm | 14mm | Rx |
|--------|-------------|-------------|-------------|-------------|-------------|-------------|----|
| 16 Fr. | IN1604-LEIR | IN1606-LEIR | IN1608-LEIR | IN1610-LEIR | IN1612-LEIR | IN1614-LEIR | • |
| 20 Fr. | IN2004-LEIR | IN2006-LEIR | IN2008-LEIR | IN2010-LEIR | IN2012-LEIR | IN2014-LEIR | • |



Large Esophageal Flange / TEP Occluder

| | 4mm | 6mm | 8mm | 10mm | 12mm | 14mm | Rx |
|--------|-------------|-------------|-------------|-------------|-------------|-------------|----|
| 16 Fr. | IN1604-LETO | IN1606-LETO | IN1608-LETO | IN1610-LETO | IN1612-LETO | IN1614-LETO | • |
| 20 Fr. | IN2004-LETO | IN2006-LETO | IN2008-LETO | IN2010-LETO | IN2012-LETO | IN2014-LETO | • |

■ Increased Resistance



Increased Resistance

| | 4mm | 6mm | 8mm | 10mm | 12mm | 14mm | Rx |
|--------|-----------|-----------|-----------|-----------|-----------|-----------|----|
| 16 Fr. | IN1604-IR | IN1606-IR | IN1608-IR | IN1610-IR | IN1612-IR | IN1614-IR | • |
| 20 Fr. | IN2004-IR | IN2006-IR | IN2008-IR | IN2010-IR | IN2012-IR | IN2014-IR | • |

■ TEP Occluder



TEP Occluder

| | 4mm | 6mm | 8mm | 10mm | 12mm | 14mm | Rx |
|--------|-----------|-----------|-----------|-----------|-----------|-----------|----|
| 16 Fr. | IN1604-TO | IN1606-TO | IN1608-TO | IN1610-TO | IN1612-TO | IN1614-TO | • |
| 20 Fr. | IN2004-TO | IN2006-TO | IN2008-TO | IN2010-TO | IN2012-TO | IN2014-TO | • |

Handsfree Speech

In 1982, Drs. Blom, Singer and Hamaker introduced the first tracheostoma valve for hands-free speech. Today, the Blom-Singer Adjustable Tracheostoma Valve (ATSV II), is a vital component in tracheoesophageal voice restoration following total laryngectomy. Commonly referred to as our 'Hands-free valve' it eliminates the need to cover the stoma with a thumb or finger to produce voice. An optional Blom-Singer HumidiFilter® Cap may be worn with the Hands-free valve to help reduce mucus secretions, coughing and dryness.



Blom-Singer Adjustable Tracheostoma Valve II (ATSV II) Starter Kit with HumidiFilter Cap

The Starter Kit includes one complete ATSV II; six Blom-Singer TruSeal Contour adhesive housings; one HumidiFilter Cap and seven HumidiFilter foam filters; two standard tracheostoma valve housings; one standard size tape/foam disc sampler pack; one 2 oz. bottle of silicone adhesive with brush cap, Skin Prep and Remove.

| REF | DESCRIPTION | Rx |
|---|--|----|
|  BE8024H | ATSV II Starter Kit w/HumidiFilter Cap | • |



ATSV II Valve
Diaphragm/Faceplate

Blom-Singer Adjustable Tracheostoma Valve II

The Adjustable Tracheostoma Valve II (ATSV II) consists of two pieces, a Diaphragm/Faceplate and Body. The assembled two-piece valve inserts into the tracheostoma adhesive housing that is applied to the skin surrounding the stoma. The Diaphragm/Faceplate is available separately.

| REF | DESCRIPTION | Rx |
|---|--|----|
|  BE8025H | ATSV II Valve (Body and Diaphragm/Faceplate) | • |
|  BE8026H | ATSV II Replacement Diaphragm/Faceplate | • |



Blom-Singer ATSV II HumidiFilter® Cap and 7 Foam Filters

The ATSV II HumidiFilter Cap is a twist-on plastic filter cap that fits over the ATSV II. It is designed to provide the laryngectomee with warmed, humidified, and filtered air during pulmonary exchange. Comes with seven HumidiFilter foam filters. (For use with ATSV II only.)

| REF | DESCRIPTION | Rx |
|--|---|----|
|  BE1010 | ATSV II HumidiFilter Cap and 7 Foam Filters | • |



ATSV II HumidiFilter Replacement Foam Filters

Replacement foam filters that fit only the ATSV II HumidiFilter Cap.

| REF | DESCRIPTION |
|--|--|
|  BE1020 | ATSV II HumidiFilter Cap Foam Filters, 7 ea |
|  BE1030 | ATSV II HumidiFilter Cap Foam Filters, 30 ea |

LEGEND

Voice prosthesis color designations denote compatibility with other products.

 Blom-Singer **Advantage**

 Blom-Singer **Dual Valve**

 Blom-Singer **Classic**

 Blom-Singer **Low Pressure**

 Blom-Singer **Duckbill**

Rx

Products with a Rx notation require a USA prescription from a qualified medical professional

Heat & Moisture Exchange—HME

The HME system helps users improve their quality of life by serving as a partial replacement for lost nasal functions. It does so by filtering and exchanging cool, dry air for warm and moist air. It's critical for laryngectomees to breathe in air that contains the proper heat and moisture content to protect the trachea and lower airway from drying and cooling. Daily, consecutive use of the Blom-Singer HME System contributes to a healthier respiratory tract, restores airflow resistance, and helps to maintain optimal lung ventilation.



The **EasyTouch™** speech button gives immediate tactile response. You'll feel the difference.



*Healthier breathing,
less mucous and coughing.*

The HME System is placed over the tracheostoma for inhaled and exhaled air to pass through. However, it does not obstruct the user's ability to breathe or talk.



Blom-Singer HME Cartridge

The HME cartridge is a lightweight, daily disposable device that inserts into all styles of the TruSeal® Adhesive Housings. The Blom-Singer HME system may be used by all laryngectomees: esophageal speakers, electrolarynx users, and tracheoesophageal prosthesis users with manual occlusion.

| REF | DESCRIPTION |
|--------|----------------------|
| BE1055 | HME Cartridge, 30 pk |

Blom-Singer HME System 7-Day Trial Kit

The Blom-Singer HME trial kit is supplied in a convenient 7 day supply. Each kit contains seven HME cartridges, seven TruSeal Adhesive Housings, and instruction manual.

| REF | DESCRIPTION |
|---------|---|
| BE1055s | Blom-Singer HME System 7-Day Trial Kit, 1 Kit |



Standard



Oval



Round

Blom-Singer TruSeal® Adhesive Housing

The TruSeal Adhesive Housing is a lightweight, disposable, all-inclusive adhesive tape-disc/housing that holds the HME cartridge. Available in two styles, Standard and Contour.

| REF | DESCRIPTION |
|--------|---|
| BE6053 | TruSeal Adhesive Housing, 7 ea |
| BE6054 | TruSeal Adhesive Housing, 30 ea |
| BE6565 | TruSeal Adhesive Housing, 500 ea |
| BE6070 | TruSeal Contour Adhesive Housing, Round, 30 ea |
| BE6071 | TruSeal Contour Adhesive Housing, Oval, 30 ea |
| BE6072 | TruSeal Contour Adhesive Housing, Standard, 30 ea |
| BE6073 | TruSeal Contour Adhesive Housing, Sample, 2 of each shape |

LEGEND

Voice prosthesis color designations denote compatibility with other products.

-  Blom-Singer **Advantage**
-  Blom-Singer **Dual Valve**
-  Blom-Singer **Classic**
-  Blom-Singer **Low Pressure**
-  Blom-Singer **Duckbill**

Rx


Products with a Rx notation require a USA prescription from a qualified medical professional

Laryngectomy Tubes & Buttons



The Blom-Singer Laryngectomy Tube is a durable, lightweight silicone tube that is used to keep the tracheostoma open after laryngectomy. It is available in a variety of sizes/styles and flexibly contours to the postoperative tracheal anatomy. For use with a voice prosthesis, the recessed area may be modified (fenestrated) by a trained clinician for use with a Blom-Singer voice prosthesis. However, for convenience a pre-fenestrated series is available that eliminates the need for manual fenestration. Blom-Singer Laryngectomy Tubes are available in three product offerings described below.

| | Size/Length | I.D./O.D. | Sterile | Nonsterile | Fenestrated Nonsterile | Rx |
|--|-------------|---------------|---------|------------|------------------------|----|
| | 8/36mm | 9.5mm/12.0mm | BE6398 | BE6298 | BE6298F | • |
| | 8/55mm | 9.5mm/12.0mm | BE6399 | BE6299 | BE6299F | • |
| | 9/36mm | 10.5mm/13.5mm | BE6400 | BE6300 | BE6300F | • |
| | 9/55mm | 10.5mm/13.5mm | BE6401 | BE6301 | BE6301F | • |
| | 10/36mm | 12.0mm/15.0mm | BE6402 | BE6302 | BE6302F | • |
| | 10/55mm | 12.0mm/15.0mm | BE6403 | BE6303 | BE6303F | • |
| | 12/36mm | 13.5mm/17.0mm | BE6404 | BE6304 | BE6304F | • |
| | 12/55mm | 13.5mm/17.0mm | BE6405 | BE6305 | BE6305F | • |

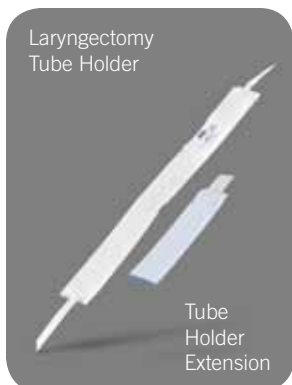

Laryngectomy Tube Holder

Designed to provide secure positioning of the Blom-Singer Laryngectomy Tube, this two-piece moisture resistant, cotton-lined neckband attaches to the laryngectomy tube flanges with easy-to-use fastener tabs. The length of the cloth strap, not including fastener tabs, is approximately 18 inches (45.7cm). Latex free. One size. Extension strap also available.

| | REF | DESCRIPTION |
|--|-----|---|
| | 240 | Tube Holder - Dale® 240 Blue with moisture resistant band, 14" (35.6cm) Long with 5" (12.7cm) Extension |
| | 243 | Tube Holder Extension - Dale® 243 Blue, 6" (15.2cm) Long |



Barton-Mayo™ Tracheostoma Button

Developed in association with the Mayo Clinic to provide leak proof tracheostoma valve retention for near-total laryngectomy and total laryngectomy patients. It is compatible with the Blom-Singer Adjustable Tracheostoma Valve II and Blom-Singer HumidiFilter.

| | REF | SIZE | O.D. | LENGTH | Rx |
|--|--------------|--|--------|---------------|----|
| | BMB091 | 9 | 13.5mm | 15mm, short | • |
| | BMB092 | 9 | 13.5mm | 22mm, regular | • |
| | BMB093 | 9 | 13.5mm | 30mm, long | • |
| | BMB101 | 10 | 15mm | 15mm, short | • |
| | BMB102 | 10 | 15mm | 22mm, regular | • |
| | BMB103 | 10 | 15mm | 30mm, long | • |
| | BMB121 | 12 | 17mm | 15mm, short | • |
| | MB122 | 12 | 17mm | 22mm, regular | • |
| | BMB123 | 12 | 17mm | 30mm, long | • |
| | BMB141 | 14 | 19mm | 15mm, short | • |
| | BMB142 | 14 | 19mm | 22mm, regular | • |
| | BMB143 | 14 | 19mm | 30mm, long | • |
| | BMBKIT | Sizing Kit* (all 12 sizes) | | | • |
| | BMBKIT-SHORT | Patient Sizing Kit-Short* (4 short sizes only) | | | • |



LEGEND

Voice prosthesis color designations denote compatibility with other products.

- Blom-Singer Advantage
- Blom-Singer Dual Valve
- Blom-Singer Classic
- Blom-Singer Low Pressure
- Blom-Singer Duckbill

Rx

Products with a Rx notation require a USA prescription from a qualified medical professional

Speech Aids

Electrolarynx speech consists of a speaking aid device that, when held against the neck or cheek, produces sound vibrations that are conducted through the acoustics of the throat and mouth. This sound is articulated by the tongue, lips and teeth into speech.



Servox® Digital Electrolarynx

The SERVOX Digital has been designed to help the laryngectomee quickly regain the ability to speak as easily and clearly as possible. With the SERVOX Digital, a clearer sound is made possible by a “digital” processor instead of an “analog” processor. Two completely independent tones are separately programmable for different volume intensities and frequency settings. There are no distracting clicking sounds of tone buttons when they are depressed. The SERVOX Digital Speech Aid complete set includes one SERVOX Digital speech aid, safety lanyard, holster, 110V charger, oral connector with two tubes, soft-tone screw cap, CD-ROM instructional software, instruction manual, and two NiMH batteries. The Software Communication Set is sold separately.

| REF | DESCRIPTION |
|--------|---|
| 17140 | SERVOX Digital Speech Aid, (2 NiMH bateries included) |
| 17216 | SERVOX Pro Demo Speech Aid (Professional NETWORK members only, limit 1 per facility) |
| 17243 | SERVOX Digital Dual Battery Charger, 110v |
| 17244 | SERVOX Digital Dual Battery Charger, 230v |
| 12174 | SERVOX Power Cord, 110v |
| 12175 | SERVOX Power Cord, 230v |
| 18253 | SERVOX Oral Connector with 2 tubes |
| 19632 | SERVOX Digital Battery |
| 17064 | SERVOX Digital Battery End Cap. |
| 17057 | SERVOX Digital Top Cap |
| 16555 | SERVOX Digital Holster |
| 17093 | SERVOX Digital Safety Cord |
| BE7279 | SERVOX Digital Oral Tubing, 25 ea |
| BE8490 | SERVOX Digital Rubber “O” Ring |



ChatterVox™ with Headset Microphone

The ChatterVox portable voice amplification system is designed to elevate vocal output to achieve a louder speech level that is comfortable and effective for everyday communication. It can boost vocal output for esophageal speakers and users of tracheoesophageal voice prostheses. ChatterVox features a Nickel-Metal Hydride (NiMH) rechargeable power system that can provide up to 14 hours of continuous use. Comes with a headset microphone and amplifier, which is normally worn at the front of the waist. Set includes: one ChatterVox voice amplifier, one ChatterVox headset microphone, one power adapter, one amplifier cover, and six rechargeable NiMH batteries.

| REF | DESCRIPTION |
|--------|--|
| BE7700 | ChatterVox™ Portable Voice Amplification System (6 NiMH batteries included) |

LEGEND

Voice prosthesis color designations denote compatibility with other products.



Blom-Singer **Advantage**



Blom-Singer **Dual Valve**



Blom-Singer **Classic**



Blom-Singer **Low Pressure**



Blom-Singer **Duckbill**

Rx


Products with a Rx notation require a USA prescription from a qualified medical professional

Blom-Singer® Voice Prosthesis Accessories



Blom-Singer Flushing Device and Cleaning Brush

These accessories are an integral part of daily maintenance. The Flushing device allows a voice prosthesis to remain in the tracheoesophageal passage while routine cleaning is performed. It is filled with water, which is then carefully injected through the voice prosthesis, to flush out accumulated debris. The cleaning brush has important safety features to prevent it from being inserted too far into the prosthesis or trachea.

| REF | DESCRIPTION | Rx |
|--|--|----|
|  IN4050 | Flushing Device, 3 ea | |
|  IN4064 | Cleaning Brush, 3 ea* 16/20 Fr. Small, 4mm–8mm | • |
|  IN4065 | Cleaning Brush, 3 ea* 16/20 Fr. Medium, 9mm–16mm | • |
|  IN4066 | Cleaning Brush, 3 ea* 16/20 Fr. Large, 18mm–22mm | • |

*Fits indwelling style prostheses only (does not fit dual valve prosthesis)



Blom-Singer Plug Insert

The Plug Insert is designed to stop leakage temporarily through a voice prosthesis. Voice production is not possible while the plug insert is in use.

| REF | DESCRIPTION | Rx |
|--|--------------------|----|
|  IN4069 | 16 Fr. Plug Insert | • |
|  IN4070 | 20 Fr. Plug Insert | • |



Blom-Singer Indwelling Valved Insert

The Indwelling Valved Insert is used with the Blom-Singer 20 Fr. Classic Indwelling voice prosthesis line, sizes 8mm and above. It contains a one-way flap valve that temporarily eliminates leakage through a voice prosthesis while simultaneously allowing the patient to speak. Safety neck strap included.

| REF | DESCRIPTION | Rx |
|--|--|----|
|  BK4100 | 20 Fr. Indwelling Valved Insert, fits sizes 7mm–25mm | • |



Blom-Singer Tracheoesophageal Puncture Dilator

The Tracheoesophageal Puncture Dilator is a tapered, solid silicone stent that dilates the tracheoesophageal puncture and also prevents leakage and closure of the puncture tract upon removal of a voice prosthesis. (Available in two diameters)

| REF | DESCRIPTION | Rx |
|--|--------------------------------|----|
|  BE6050 | 18 Fr. (for 16 Fr. prostheses) | • |
|  BE2050 | 22 Fr. (for 20 Fr. prostheses) | • |

LEGEND

Voice prosthesis color designations denote compatibility with other products.



Blom-Singer **Advantage**



Blom-Singer **Dual Valve**



Blom-Singer **Classic**



Blom-Singer **Low Pressure**



Blom-Singer **Duckbill**




Rx

Products with a Rx notation require a USA prescription from a qualified medical professional



Blom-Singer Gel Cap Insertion System










The Gel Cap Insertion System provides an optional method for easy and comfortable insertion of a Blom-Singer Low Pressure voice prosthesis into the tracheoesophageal passage. Each package contains a loading tool, pushrods, gel caps and lubricant. Replacement gel caps available.

| | REF | DESCRIPTION | Rx |
|---|--------|--|----|
|  | LP3100 | 16/20 Fr. Low Pressure Gel Cap Insertion System* | • |
|  | BE3190 | Replacement Gel Caps, 16 Fr., 90 ea | • |
|  | BE3290 | Replacement Gel Caps, 20 Fr., 90 ea | • |



Blom-Singer 16 Fr. TEP Occluder

The TEP Occluder maintains the puncture tract during the time that the voice prosthesis is removed. This device is not intended for voicing.

| | REF | DESCRIPTION | Rx |
|---|----------|---------------------------|----|
|  | DM16-006 | 6mm TEP Occluder, 16 Fr. | • |
|  | DM16-008 | 8mm TEP Occluder, 16 Fr. | • |
|  | DM16-010 | 10mm TEP Occluder, 16 Fr. | • |
|  | DM16-012 | 12mm TEP Occluder, 16 Fr. | • |
|  | DM16-014 | 14mm TEP Occluder, 16 Fr. | • |
|  | DM16-018 | 18mm TEP Occluder, 16 Fr. | • |
|  | DM16-022 | 22mm TEP Occluder, 16 Fr. | • |
|  | DM16-025 | 25mm TEP Occluder, 16 Fr. | • |
|  | DM16-028 | 28mm TEP Occluder, 16 Fr. | • |

Lubricants

| REF | DESCRIPTION |
|---------|--|
| 0205-37 | SurgiLube Lubricating Jelly (4.25 oz. tube) |
| T00128 | PDI® Lubricating Jelly, 2.7 gram packets, box of 144 |

LEGEND

Voice prosthesis color designations denote compatibility with other products.



Blom-Singer **Advantage**



Blom-Singer **Dual Valve**



Blom-Singer **Classic**



Blom-Singer **Low Pressure**



Blom-Singer **Duckbill**

Rx

Products with a Rx notation require a USA prescription from a qualified medical professional

Blom-Singer® Tracheostoma Attachments

To ensure the successful use of the Blom-Singer HME System or ATSV II, a full line of products are available for your convenience.

Attachment of an HME cartridge or ATSV II to the tracheostoma is achieved through the use of a TruSeal adhesive housing or tracheostoma valve housing.



HME cartridge





ATSV II with HumidiFilter Cap



Standard

Blom-Singer TruSeal® Adhesive Housing

The TruSeal Adhesive Housing is a lightweight, disposable, all-inclusive adhesive tape-disc/housing that eliminates the need for a tracheostoma valve housing, tape, or adhesive foam disc. It is designed to hold the ATSV II, HME cartridge or Shower Guard over the stoma.





| REF | DESCRIPTION |
|--|--|
|  BE6053 | TruSeal Adhesive Housing, Standard, 7 ea |
|  BE6054 | TruSeal Adhesive Housing, Standard, 30 ea |
|  BE6565 | TruSeal Adhesive Housing, Standard, 500 ea |



Oval

Blom-Singer TruSeal® Contour Adhesive Housing

The new TruSeal Contour Adhesive Housing is a lightweight, malleable material that comfortably fits various peristomal anatomies. It is offered in three shapes and will stand up to the rigorous demands of everyday use. The TruSeal Contour is a disposable, all-inclusive adhesive tape-disc/housing that eliminates the need for a tracheostoma valve housing, tape, or adhesive foam disc. It is designed to hold the ATSV II, HME cartridge or Shower Guard over the stoma.

| REF | DESCRIPTION |
|--|---|
|  BE6070 | TruSeal Contour Adhesive Housing, Round, 30 ea |
|  BE6071 | TruSeal Contour Adhesive Housing, Oval, 30 ea |
|  BE6072 | TruSeal Contour Adhesive Housing, Standard, 30 ea |
|  BE6073 | TruSeal Contour Adhesive Housing, Sample, 2 of each shape |



Round



Tracheostoma Valve Housing

Valve Housing, PVC

Tracheostoma Valve Housing

The Tracheostoma Valve Housing is a reusable, flexible housing that attaches the ATSV II, HME cartridge or Shower Guard over the stoma. It is applied with separate adhesive tape discs and silicone adhesive. It is available in two sizes, standard and large.

| REF | DESCRIPTION |
|--|---|
|  BE6038 | Tracheostoma Valve Housing, standard, 1 ea |
|  BE6039 | Tracheostoma Valve Housing, large, 1 ea |
|  BE6038-P | Tracheostoma Valve Housing, standard, PVC, 1 ea |
|  BE6039-P | Tracheostoma Valve Housing, large, PVC, 1 ea |

LEGEND

Voice prosthesis color designations denote compatibility with other products.



Blom-Singer **Advantage**



Blom-Singer **Dual Valve**



Blom-Singer **Classic**



Blom-Singer **Low Pressure**



Blom-Singer **Duckbill**

Rx

Products with a Rx notation require a USA prescription from a qualified medical professional



Adhesive Discs

A variety of double-sided adhesive foam and tape discs are available to provide the optimal attachment of the tracheostoma valve housing to your stoma.

Adhesive Foam Discs

| REF | DESCRIPTION |
|--|-------------------------------------|
|  BE6045 | Regular Foam Discs, standard, 30 ea |
|  BE6046 | Regular Foam Discs, large, 30 ea |
|  BE6047 | Thin Foam Discs, standard, 30 ea |
|  BE6049 | Thin Foam Discs, large, 30 ea |

Adhesive Tape Discs

| REF | DESCRIPTION |
|--|--|
|  BE6041 | Regular Tape Discs, standard, 30 ea |
|  BE6042 | Regular Tape Discs, large, 30 ea |
|  BE6043 | Thin Tape Discs, standard, 30 ea |
|  BE6044 | Thin Tape Discs, large, 30 ea |
|  BE6034 | Heavy Duty Tape Discs, standard, 30 ea |
|  BE6035 | Heavy Duty Tape Discs, large, 30 ea |

Adhesive Foam and Tape Disc Sampler Pack

Sample pack of five each of the standard size Foam and Tape Discs.

| REF | DESCRIPTION |
|--|---|
|  BE6051 | Adhesive Foam and Tape Disc Sampler Pack, standard size |



Blom-Singer® Silicone Adhesive

Safely attaches appliances and tape to skin. Easily removed with Remove™. Available in a 2 oz. bottle with brush cap.

| REF | DESCRIPTION |
|--|---|
|  BE6067 | Blom-Singer Silicone Adhesive, 2 oz. bottle |



Skin Tac® Wipes



Skin adhesive aids in the adhesion of appliances and tape. An alternative to bottled liquid adhesive. Ideally packaged for travel.

| REF | DESCRIPTION |
|--|---|
|  MS407W | Skin Tac Wipes (skin adhesive), box of 50 |



3M MicroPore™ Tape

Hypoallergenic and porous, 3M MicroPore Tape is gentle to the skin. This tape provides the ability to quickly fix a leak that may occur periodically in tracheostoma attachment. 3M Tape is also convenient for taping a voice prosthesis strap to the neck.

| REF | DESCRIPTION |
|--|--------------------------------------|
|  BE6056 | 3M MicroPore Tape, 1" (2.5cm) Roll |
|  BE6057 | 3M MicroPore Tape, 1/2" (1.3cm) Roll |

LEGEND

Voice prosthesis color designations denote compatibility with other products.



Blom-Singer **Advantage**



Blom-Singer **Dual Valve**



Blom-Singer **Classic**



Blom-Singer **Low Pressure**



Blom-Singer **Duckbill**

Rx

Products with a Rx notation require a USA prescription from a qualified medical professional

Skin Care

Adhesive Barriers *Ideal for protecting the skin around the stoma from adhesives.*



Shield Skin®

Provides a clear, protective, plasticized barrier between skin and external devices for better tape and adhesion. Water-resistant and non-staining.

| REF | DESCRIPTION |
|-------|---------------------------|
| 44700 | Shield Skin, 2 oz. bottle |
| 44005 | Shield Skin, box of 50 |



Skin-Prep™

Effective topical barrier between skin and external devices for better tape adhesion. Non-irritating.

| REF | DESCRIPTION |
|---------|----------------------|
| 4204-00 | Skin-Prep, box of 50 |



Cavilon™ No Sting Barrier Film

Convenient pad protects skin from fluids, adhesives, and friction. No sting formula dries quickly to form a breathable, transparent coating.

| REF | DESCRIPTION |
|------|--|
| 3344 | Cavilon No Sting Barrier Film, box of 25 |

Adhesive Removers *Ideal for removing adhesive from around the stoma when changing the attachments for the HME, ATSV II, HumidiFilter Holder, or Shower Guard.*



Remove™

Solvent formulated to dissolve adhesives and assist in removing residues from the skin. Convenient, easy-to-carry, single-use wipes.

| REF | DESCRIPTION |
|---------|---------------------------------------|
| 4031-00 | Remove™ Adhesive Remover Wipes, 50 ea |



TacAway™

A convenient, easy to use, one-step adhesive remover. Unique formula effectively removes adhesive residue, leaving the skin clean and dry. No greasy film.

| REF | DESCRIPTION |
|--------|--|
| MS408W | TacAway™ Adhesive Remover Wipes, 50 ea |



Alcohol Prep Pads

Removes oils from skin, ideal for preparation of stoma area for the attachment of the HME, ATSV II, Showerguard, and HumidiFilter. Also ideal for removing Skin-Tac Adhesive Wipe.

| REF | DESCRIPTION |
|--------|-------------------------------|
| C29600 | Alcohol Prep Pads, box of 100 |

Sanitizers *Ideal for maintaining a clean contact for the device and protects against bacteria.*



Norisc® Germicidal Towelettes

Conveniently maintains clean contact from hand to device. Rinseless formula offers same benefits as antimicrobial soap. Perfect for travelling or when standard hand-washing facilities are not available. Enriched with Aloe Vera.

| REF | DESCRIPTION |
|-------|---|
| 10903 | Norisc Germicidal Towelettes, bag of 30 |



Ca-Rezz® Cream

Formulated with Triclosan and Nonoxynol-9, ensures a long lasting antimicrobial barrier, protecting against bacteria, yeasts and fungi. Vitamins A, D, & E create a gentle, healing formula that leaves the skin feeling healthy and soft. Allantoin and lanolin protect and soothe while offering quick relief to sore, sensitive areas.

| REF | DESCRIPTION |
|-------|---------------------------|
| 10204 | Ca-Rezz Cream, 4 oz. tube |

LEGEND

Voice prosthesis color designations denote compatibility with other products.



Blom-Singer **Advantage**



Blom-Singer **Dual Valve**



Blom-Singer **Classic**



Blom-Singer **Low Pressure**



Blom-Singer **Duckbill**

Rx

Products with a Rx notation require a USA prescription from a qualified medical professional

Introducing... The New Blom-Singer® Scar Management Gel Sheet

- *Self-adhering*
- *Reusable*
- *Washable*

Now Available in Two Shapes:



CIRCLE

BE6080 2.75" OD x 1" ID Circle to reduce scarring around the stoma Package of 1



RECTANGLE

BE6081 1.5" x 5" Rectangle shaped to reduce large straight scarring Package of 2

For best results, the Blom-Singer® Scar Management Gel Sheet should be applied as soon after healing as possible. However, it may be used on older scars. Regardless of the scar's age, patients should begin to see the scar soften, flatten and fade within 7 to 90 days.

Research

The following are excerpts or summaries from various professional journal articles regarding the use and effectiveness of silicone-based products like our Blom-Singer® Scar Management Products for fading scars:

Results from at least eight randomized controlled trials and a meta-study of 27 trials demonstrate that silicone gel sheeting is a safe and effective management option for hypertrophic scars and keloids.

International Clinical Recommendations on Scar Management, T. Mustoe, R. Cooter, M. Gold, R. Hobbs, A. Ramelet, P. Shakespeare, M. Stella, L. Teot, F. Wood, U. Ziegler. Plastic and Reconstructive Surgery Vol 110, No. 2 August 2002:560-571

"Silicone gel sheeting is safe and effective treatment for hypertrophic and keloid scars". All subjects applied the gel sheeting in the same fashion and wore it for twelve to twenty-

four hours per day for at least two months. After at least six months' follow-up, 56% of the chronic scars were improved. Eleven of fourteen fresh or recent hypertrophic scars (79%) did not recur after surgery during a similar follow-up period.

Therapeutics for the Clinician, Silicone Gel Sheeting in Scar Therapy, Katz B.E., Cutis 1995; 56:65-67

"We concluded that topical silicone gel is efficacious, both in prevention and in the treatment of hypertrophic scars" We studied the effects of a silicone gel bandage that was worn for at least 12 hours daily on the resolution of hypertrophic burn scar. Clinical assessment corroborated this quantitative demonstration of a decrement in scar volume.

Topical Silicone Gel for the Prevention and Treatment of Hypertrophic Scar S.T Ahn, W.H. Monafu, T.A. Mustoe, Arch Surg-Vol.126 April 1991: 499-504

Stoma Protection & Accessories

Guards and Shower Collars *Designed to protect the tracheostoma during showering. Not intended for swimming. One size, available in several styles.*



Blom-Singer Shower Guard

The Blom-Singer Shower Guard is a one-piece, right-angle plastic shower guard that comes with a Tracheostoma Standard Adhesive Housing and 30 double-sided Adhesive Tape Discs. Can also be attached to the stoma using the TruSeal Adhesive Housing.

| REF | DESCRIPTION | Rx |
|--|--------------------|----|
|  BE6048 | Shower Guard, 1 ea | • |



Rubber Shower Collar

The Rubber Shower Collar is made of soft and pliable vinyl. The adjustable Velcro® strap fits most necks.

| REF | DESCRIPTION |
|--|----------------------------|
|  BE6058 | Rubber Shower Collar, 1 ea |



Nylon Shower Collar

The lightweight Nylon Shower Collar is made of PVC and nylon. The adjustable Velcro® strap fits most necks.

| REF | DESCRIPTION |
|---|---------------------------|
|  BE6068 | Nylon Shower Collar, 1 ea |



Blom-Singer Foam Stoma Protectors



A simple and temporary (disposable) solution to prevent debris from entering the stoma. An adhesive strip on one edge allows for easy attachment, access to the stoma to occlude for speech with a voice prosthesis, or for coughing. Measurements listed below are approximate.

| REF | DESCRIPTION |
|--|---|
|  BE6200 | White, 2.4" L x 2.0" W (6cm x 5cm), 30 pk |
|  BE6210 | White, Thick 2.4" L x 2.0" W (6cm x 5cm), 30 pk |
|  BE6215 | Tan, 2.4" L x 2.0" W (6cm x 5cm), 30 pk |
|  BE6220 | Tan, Large 2.7" L x 2.7" W (7cm x 7cm), 30 pk |



Forceps (Tweezers)

High-quality stainless steel surgical forceps aid in the cleaning of hard to reach areas in and around the stoma. Available Angled or Straight.

| REF | DESCRIPTION |
|--|---|
|  15-143 | Angled Forceps (tweezers), 6¼", (16cm) |
|  BE6028 | Straight Forceps (tweezers), 5", (13cm) |



Cloth Stoma Protectors

A discreet and comfortable way to cover the stoma for men and women. Looks like a shirt and slips easily under other layers of clothing. Made of a soft 50/50 blend of cotton and rayon, machine washable, and has an adjustable strap that fits most necks. Length ranges from 47 to 53cm.

| REF | DESCRIPTION |
|--|-------------|
|  BE6201 | White |
|  BE6251 | Black |
|  BE7211 | Navy Blue |

LEGEND

Voice prosthesis color designations denote compatibility with other products.



Blom-Singer **Advantage**



Blom-Singer **Dual Valve**



Blom-Singer **Classic**



Blom-Singer **Low Pressure**



Blom-Singer **Duckbill**

Rx

Products with a Rx notation require a USA prescription from a qualified medical professional

Mouth & Stoma Moisturizers



Mouth-Kote®

Relieves dry mouth and throat for two to five hours. Derived from the Yerba Santa plant which mimics saliva, has a pleasant citrus taste, sugar-free, alcohol-free, low in sodium, and safe to swallow.

| REF | DESCRIPTION |
|--------|---|
| 098-02 | Mouth-Kote Oral Moisturizer, 2 oz. bottle |
| 098-08 | Mouth-Kote Oral Moisturizer, 8 oz. bottle |



Unit Dose Saline Solution, 15ml, .9% Sodium Chloride

Aids in secretion removal and hydration of the tracheostoma. Chemical and preservative free.

| REF | DESCRIPTION |
|--------|--|
| SA0924 | Unit Dose Saline Solution, 15ml, box of 24 |



Simply Saline™ Sterile Mist and Moisturizer

Moisturizes and clears congestion, does not burn, sting or harm delicate tissues. Naturally pure and preservative free, contains a sterile solution of 0.9% sodium chloride and purified water.

| REF | DESCRIPTION |
|-------|---|
| B2915 | Simply Saline, Moisturizer, 1.5 fl. oz. |
| B2916 | Simply Saline, Sinus & Allergy, 1.5 fl. oz. |



Vicks® Personal Steam Inhaler

Comfortable hood adapts to the stoma area to allow cool air to enter and mix with the steam to loosen mucus and phlegm from respiratory tract. Includes steam inhalant pad with eucalyptus. Dual voltage and compact size make it ideal for travel.

| REF | DESCRIPTION |
|-------|------------------------------|
| V1200 | Vicks Personal Steam Inhaler |

Vicks® Inhalant Pads

Each pad gradually releases soothing aromatic vapors when used with the Vicks Inhaler.

| REF | DESCRIPTION |
|-------|-------------------------------|
| KMP-6 | Vicks Inhalant Pads, box of 6 |

Personal Needs & Stoma Illumination



Laryngectomy Needs Kit

The kit comes in a discreet black travel bag containing a small, two-way mirror (one face magnifying), a light (requires 2 AA batteries), 5" (12.7cm) straight forceps for daily cleaning, saline solution vials for irrigating the stoma, Norisc™ antimicrobial wipes, alcohol prep pads, and a white cloth stoma cover. Also comes with the patient demonstration video by Richard Crum, InHealth consultant and experienced laryngectomee. The stylish travel bag allows ample room for other personal supplies and allows many personal stoma care items to be kept always on hand.

| REF | DESCRIPTION |
|--------|------------------------|
| PK1000 | Laryngectomy Needs Kit |



Personal Alarm with Flashlight

In emergency situations this convenient device provides three important functions: an alarm siren, a flashing light, and a flashlight. The 105 decibel siren activates by pulling the rip-cord while the flashing light to attract attention begins at the touch of a button. The statement - "My larynx has been operated on and I am unable to speak. I need HELP immediately." - is clearly printed on the device for responding personnel. Pocket size ideal for travel or when away from home.

| REF | DESCRIPTION |
|--------|--|
| BE7800 | Personal Alarm with Flashlight (9v battery included) |



Laryngectomy Lighted Mirror

This chrome lighted 5x magnification mirror from Conair is ideal for daily stoma care. The mirror has various settings for light intensity, and provides illumination that resembles evening, home, office, and daylight. The magnification ensures that the reflection of the stoma area is large and clear. The neck of the mirror angles easily to suit individual patient preference.

| REF | DESCRIPTION |
|---------|-----------------------------|
| BE18LCX | Laryngectomy Lighted Mirror |



Maglite® Solitaire

Less than an ounce and only three inches in length, this surprisingly bright light is a perfect travel companion for on-the-go stoma care. Comes with one AAA battery.

| REF | DESCRIPTION |
|--------|-------------------|
| K3A012 | Maglite Solitaire |




Zelco® Flexlight

This flashlight has a unique long reaching flexible neck to provide bright light where needed and is small enough to fit in the palm of your hand. The clip-on feature allows it to be used hands free. Extra bulb included, requires two AAA batteries.

| REF | DESCRIPTION |
|-------|-----------------|
| 90003 | Zelco Flexlight |


Professional Products

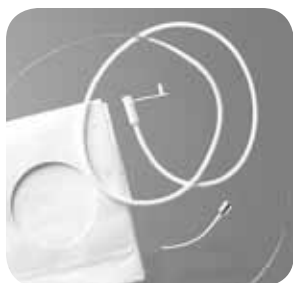

Blom-Singer Tracheoesophageal Puncture Set*

Designed for secondary placement of a tracheoesophageal puncture. Includes a disposable 16 gauge curved introducer needle with a 14 gauge Teflon® sheath, a 26" (66cm) silicone catheter with a tapered reinforced tip attached to a 24" (61cm) Teflon-coated stainless steel leader cable, and a fenestrated surgical drape with adhesive around the aperture.

| REF | DESCRIPTION |
|--------|---|
| BE6055 | Tracheoesophageal Puncture Set, Sterile |

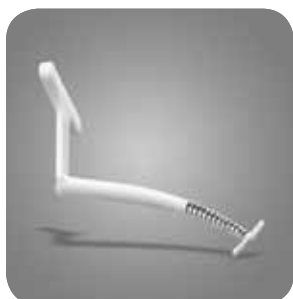

Blom-Singer Dilator-Sizer*

Why utilize two separate instruments to dilate and measure the tracheoesophageal puncture tract? The new Blom-Singer Dilator-Sizer combines two crucial tasks into one innovative product. Now, in one step the clinician can dilate and measure the tracheoesophageal passage during voice prosthesis placement procedures.

| REF | DESCRIPTION |
|--------|--|
| BE2051 | 18 Fr. Dilator-Sizer (for 16 Fr. prostheses), 1 ea |
| BE2052 | 22 Fr. Dilator-Sizer (for 20 Fr. prostheses), 1 ea |



Blom-Singer Voice Prosthesis Sizer*

The Blom-Singer Voice Prosthesis Sizer is a measurement device used by physicians and qualified speech pathologists to select the appropriate Blom-Singer voice prosthesis length. The device measures the length of tracheoesophageal puncture between the trachea and the esophagus. Markers on the device correspond to the length of the prosthesis. The tracheoesophageal puncture tract should always be measured before inserting a voice prosthesis to confirm tract length. The sizer is available in either 16 Fr. or 20 Fr. diameter. Three inserters included; replacement inserters available separately.

| REF | DESCRIPTION |
|--------|--|
| BE6036 | Voice Prosthesis Sizer, 16 Fr., with 3 inserters, 1 kit |
| BE2036 | Voice Prosthesis Sizer, 20 Fr., with 3 inserters, 1 kit |
| BE6096 | Replacement Inserters For use with both 16 Fr. and 20 Fr. sizers, 10 each |




Manometer with Manometer Adaptor*

A training aid for clinicians to teach patients how to self-regulate intratracheal air pressure. Proper airflow control results in better voice quality, less fatigue, and longer duration of the tracheostoma adhesive housing seal.

| REF | DESCRIPTION |
|--------|--|
| BE8000 | Manometer Kit includes one manometer, one manometer adaptor and one TruSeal adhesive housing |
| BE8020 | Replacement Manometer Adaptor includes 5 manometer adaptors and 5 TruSeal adhesive housings |

* Restricted to sale or use by a licensed, medical professional.

Professional Products *(continued)*

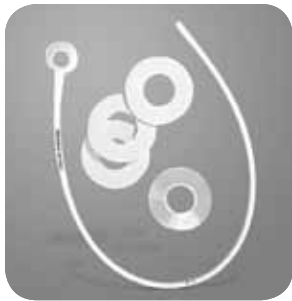

Blom-Singer Insufflation Test Set*

Aids the physician or speech pathologist in administering the esophageal insufflation test to determine the ability of a laryngectomized patient to produce tracheoesophageal voice. Disposable system consists of a 50cm long, silicone (latex-free), 14 French catheter imprinted with a 25cm length marker attached to an insertable tracheostoma adapter. The tip of the catheter is radiopaque for convenient fluoroscopic viewing. A tracheostoma housing and double-sided adhesive discs are provided for attaching the tracheostoma adapter.

| REF | DESCRIPTION |
|--------|---------------------------------|
| BE6040 | Insufflation Test Set, box of 6 |



Scissors*

Assists clinician in removal of the silicone neck strap on a Blom-Singer Indwelling Voice Prosthesis.

| REF | DESCRIPTION |
|--------|---|
| BE6062 | 7½" (19cm) curved, angled, flat, serrated |



Hemostat*

Assists clinician in the withdrawal of an indwelling-style voice prosthesis in routine maintenance.

| REF | DESCRIPTION |
|--------|------------------------------|
| BE6061 | 5" (12.7cm) straight locking |



Clinician Headlight

Used for examining the tracheostoma, the compact, lightweight design and comfortable fit make this cordless headlight well-suited for otolaryngologists, speech pathologists, and other medical staff. Designed with portability and economy in mind. Two AA batteries are included. Replacement Bulbs and Replacement Headband are available separately.

| REF | DESCRIPTION |
|--------|--|
| BE9100 | Clinician Headlight |
| BE9150 | Clinician Headlight Replacement Bulb |
| BE9160 | Clinician Headlight Replacement Headband |

Headlight Specifications

| | |
|--------------|-------------------------------|
| Lamp Type | Xenon |
| Battery Burn | Time 5-6 hours |
| Lamp Life | 20 hours |
| Battery Type | E91 1.5V Alkaline AA-cell (2) |

LEGEND

Voice prosthesis color designations denote compatibility with other products.

-  Blom-Singer **Advantage**
-  Blom-Singer **Dual Valve**
-  Blom-Singer **Classic**
-  Blom-Singer **Low Pressure**
-  Blom-Singer **Duckbill**

Rx

Products with a Rx notation require a USA prescription from a qualified medical professional

* Restricted to sale or use by a licensed, medical professional.


ENT Products



Oto-Rhino Foreign Body Remover*

The Katz Extractor Oto-Rhino Foreign Body Remover is a small, lightweight, single-use device for quick and easy extraction of foreign bodies from the nose or external ear canal. It comes in a convenient, ready-to-use five-pack box. There are no expensive supplies (such as Fogarty's, Foleys, syringes, forceps, etc.) to hunt and assemble, and no additional suction equipment is required.

| |
|------------------|
| Ready-to-use |
| Atraumatic |
| Quick Extraction |
| Economical |

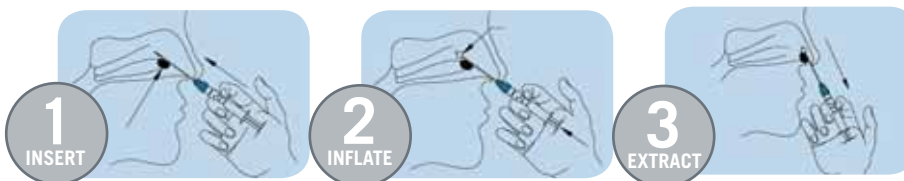

Katz Extractor Oto-Rhino Foreign Body Remover

Removal of foreign bodies has never been easier. It takes three simple steps to complete an extraction. During trial studies, most extractions took, on average, less than 2 minutes to complete.

| REF | DESCRIPTION |
|---------|---|
| FB 5000 | Katz Extractor Oto-Rhino Foreign Body Remover, box of 5 |



Freeman® Frontal Sinus Stent* with Blom-Singer® Gel Cap Insertion System

The Freeman Frontal Sinus Stent is designed to provide post-operative drainage of the frontal sinus and to prevent blockage of the frontal outflow tract following sinus surgery. The stent is intended for placement with an insertion instrument under endoscopic or direct visualization. Incorporating the Blom-Singer gel cap insertion system, the stent can be inserted with the use of a gel cap. This silicone stent includes angled flanges to provide retention. Provided as a sterile kit and includes: one stent, one insertion instrument, one Blom-Singer gel cap loading tool, and three gel caps.

| REF | DESCRIPTION |
|------------|------------------------------|
| BE 1400-R2 | 14 Fr, diameter, 20mm length |
| BE 1600-R2 | 16 Fr, diameter, 20mm length |



Freeman Frontal Sinus Stent DVD

Narrated by Stephen B. Freeman, M.D., F.A.C.S. This 16 minute video includes discussion of the objectives of the stent, indications for use, insertion procedures, including endoscopic procedure and external approach, as well as removal of the stent. (Mac and PC compatible)

| REF | DESCRIPTION |
|--------|-----------------------------|
| CD9000 | Freeman Frontal Sinus Stent |

* Restricted to sale or use by a licensed, medical professional.

Educational Videos and Books



Surgical Voice Restoration DVD

Narrated by Ronald C. Hamaker, M.D., F.A.C.S., includes primary TEP, secondary TEP, myotomy, neurectomy, tracheostoma construction, secondary pharyngeal constrictor muscle myotomy, and botulinum toxin chemical neurectomy. 30 minutes.

| REF | DESCRIPTION |
|--------|--|
| CD3000 | Tracheoesophageal Voice Restoration After Total Laryngectomy |



Speech Restoration DVD

Demonstration of tracheoesophageal voice restoration. Narration by Eric D. Blom, Ph.D. 30 minutes.

| REF | DESCRIPTION |
|--------|-------------------------------|
| CD6000 | Blom-Singer Voice Restoration |



Adjustable Tracheostoma Valve Patient Demonstration DVD

Richard Crum illustrates and discusses daily maintenance of handsfree speech, with the ATSV II and HME System.

| REF | DESCRIPTION |
|--------|----------------------------|
| CD8000 | ATSV Patient Demonstration |



Educational Books

| REF | DESCRIPTION |
|--------|--|
| TA5000 | Self-Help for the Laryngectomee, by Edmund Lauder |
| TA7000 | Looking Forward...The Speech and Swallowing Guidebook for People with Cancer of the Larynx or Tongue by Jack E. Thomas and Robert L. Keith |



Post Laryngectomy Training Aid Set

These detailed anatomical diagrams have been used by Otolaryngologists and Speech Pathologists worldwide and are ideally suited for teaching other professionals or patients. This full color set includes five laminated anatomical drawings illustrating pre-laryngectomy anatomy; post-laryngectomy anatomy; esophageal speech; electrolarynx use; and tracheoesophageal voice prosthesis. Also available in a poster or sketch pads. Pictured at left.

| REF | DESCRIPTION |
|--------|------------------------------------|
| TA1000 | Post-laryngectomy Training Aid Set |

Training Aids



Training Aid Poster

A full color poster, 18" x 24" (46cm x 61cm), for clinic or office, illustrating anatomical depictions of Before and After Laryngectomy; Electrolarynx Speech; Esophageal Speech; and, Tracheoesophageal Speech. Beneficial in helping patients to understand voice restoration options.

| REF | DESCRIPTION |
|--------|---------------------|
| TA1010 | Training Aid Poster |



Training Aid Sketch Pads

A series of sketch pads (50 sheets to each pad) showing anatomical illustrations depicted in the Training Aid Poster. In full color with room for notes at the bottom.

| REF | DESCRIPTION |
|--------|---|
| TA1020 | Before Laryngectomy Sketch Pad |
| TA1021 | After Laryngectomy Sketch Pad |
| TA1022 | Esophageal Speech Sketch Pad |
| TA1023 | Artificial Larynx Sketch Pad |
| TA1024 | Tracheoesophageal Voice Prosthesis Sketch Pad |



Laryngectomy Needs Chart

Created with the assistance of the laryngectomee community, this laminated chart assists new laryngectomees with communications during the days after surgery. This chart is available in English or Spanish. Call today to request this complimentary chart for your club or for your hospital to stock.

| DESCRIPTION |
|---|
| Laryngectomy Needs Chart (English & Spanish Only) |

Ordering Information

USA Customers

Our caring and knowledgeable customer service representatives are available to assist you, Monday through Thursday, 9:30 am to 7:00 pm, and on Friday, 9:30 am to 5:00 pm, Eastern Standard Time. Our toll-free number may be used from the continental USA, Alaska, Hawaii, Puerto Rico, and the Virgin Islands.

Ordering Information

We at InHealth Technologies strive to provide the method of ordering that is most convenient for you. Please choose from the following:

Telephone Toll-Free (800) 477-5969 or (805) 684-9337
Fax Toll-Free (888) 371-1530 or (805) 684-8594
E-Mail order@inhealth.com
Order Online www.inhealth.com
Mail InHealth Technologies
1110 Mark Avenue, Carpinteria, CA 93013-2918 USA

Shipping and Handling

Standard shipping is a 3-business day service. Overnight service is available upon request at an additional charge. Due to Air Safety Standards, orders containing Simply Saline will be shipped via Ground (5 to 7 business days). Please contact customer service for details.

| ZONE | GROUND | STANDARD (3 BUSINESS DAYS) | 2ND DAY | OVERNIGHT | SATURDAY | STATES |
|-------|------------|-------------------------------|----------------------------|----------------------------|----------------------------|---|
| One | Add \$6.50 | Add \$9.90 COD \$18.90 | Add \$14.50 COD \$23.50 | Add \$17.50 COD \$26.50 | Add \$30.00 COD \$39.00 | AZ, CA, CO, ID, KS, NM, MT, NV, OK, OR, TX, UT, WA, WY |
| Two | Add \$7.50 | Add \$11.50 COD \$20.50 | Add \$15.50 COD \$24.50 | Add \$21.50 COD \$30.50 | Add \$34.00 COD \$43.00 | AL, AR, IA, IL, LA, IN, KY, MI, MN, MO, MS, ND, NE, OH, SD, TN, WI |
| Three | Add \$8.00 | Add \$13.00 COD \$22.00 | Add \$17.50 COD \$26.50 | Add \$22.50 Add \$31.50 | Add \$35.00 COD \$44.00 | CT, DC, DE, FL, GA, MA, MD, ME, NC, NH, NJ, NY, PA, RI, SC, VA, VT, WV |
| Four | N/A | N/A | Add \$27.00 COD \$36.00 | Add \$30.50 Add \$39.50 | Add \$43.00 COD \$52.00 | AK, HI, PR |

Shipping rates are subject to change

Payment

Check, Money Order, VISA, MasterCard, Discover, American Express, and C.O.D. are accepted.

Special Products

Please contact our customer service department for full ordering details at (800) 477-5969 or email us at info@inhealth.com

Prescription Devices

Products with Rx next to their name are prescription items. Please note that a prescription from a qualified medical professional must be on file with InHealth Technologies for these products (individual Rx items must be specifically listed on the prescription). Caution: USA Federal law restricts these devices to sale by or on the order of a qualified medical professional.

HIPAA Readiness Statement

InHealth Technologies values your trust and the relationships we share – so we take great care to inform you of our efforts to conform to all applicable HIPAA regulations covering “Privacy Standards.” As a business charged with the responsibility of receiving, processing, distributing and storing “private health information” (PHI), our business procedures and computer systems have been modified to incorporate compliance requirements. In some cases, InHealth may disclose health information only for the purposes of facilitating payment of patient invoices for our clients. Prior to disclosing such information InHealth must first receive a signed HIPAA Consent Agreement from the patient. These agreements will assure all parties that individual healthcare information privacy rights are maintained to safeguard patient information.

Medicare

A Medicare information form is available upon request. Please contact our customer service department for a copy of the form. If a patient is a Medicare beneficiary, we will need a completed Medicare information form and a copy of the beneficiary’s Medicare insurance card. This form need only be submitted one time, unless information such as address or eligibility status has changed. Once current information is on file, Medicare claims will be submitted on a routine monthly basis. All orders must be prepaid — we do not accept Medicare assignment. Medicare reimbursement will come directly to the patient.

Insurance Reimbursement

Two copies of your purchase receipt will be mailed to you after each shipment/purchase. These receipts may be included in the claim to a private insurance carrier.

Consumer Affairs

If you have any questions or dissatisfaction with a product, please contact our customer service department at (800) 477-5969 or you may email us at consumeraffairs@inhealth.com

Returned Goods Policy

All returned merchandise must have a Return Merchandise Authorization number (RMA), and must be unopened and undamaged. Returns without an RMA will not be accepted. Products must be returned in unopened packages, with manufacturer’s tamper evident seals intact to be accepted for replacement or credit. Products will not be accepted for replacement or credit if they have been in the possession of the customer for more than 14 days. Special order products are not returnable. There is a 20% restocking fee on all returned merchandise. To obtain an RMA number, please contact us at (800) 477-5969 or email us at productreturns@inhealth.com

Additional Order Forms

If you would like an additional order form or would like one sent to a colleague or friend, please call customer service at (800) 477-5969 or email your request at info@inhealth.com

International Customers

InHealth products are available through authorized distributors worldwide. For international distributor information, please contact our customer service department.

LIMITED WARRANTY

InHealth Technologies, a division of Helix Medical, LLC (Seller) warrants that the Product will perform substantially in accordance with the accompanying instructions and will be free from defects in materials and workmanship under normal use for a period of fourteen (14) days from date of receipt. Seller's entire liability and your exclusive remedy shall be, at Seller's option, either (a) return of the price paid, or (b) repair or replacement of the Product that does not meet Seller's Limited Warranty and that is returned with a copy of the receipt. No returns will be made without authorization from Seller by calling 1-800-477-5969. This Warranty may be void if the Product is damaged by improper storage, accident, misapplication, or if it is modified or subject to misuse or abuse.

TO THE EXTENT PERMITTED BY LAW, THE FOREGOING WARRANTIES ARE EXCLUSIVE AND IN LIEU OF ALL OTHER WARRANTIES OR REPRESENTATIONS, EXPRESS OR IMPLIED, ARISING BY LAW OR CUSTOM, INCLUDING IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, SAID IMPLIED WARRANTIES BEING HEREBY EXPRESSLY DISCLAIMED.

To the extent permitted by law, in no event shall Seller be liable for any direct, indirect, special, incidental, consequential, punitive or exemplary damages of any kind, even if Seller has been advised of the possibility of such damages. These limited warranty terms give you specific legal rights, and some states/jurisdictions do not allow limitations on the duration of an implied warranty or the exclusion or limitation of liability for certain damages, therefore the above limitations, in whole or in part, may not apply to you.

The products depicted in this catalog may be covered by, but not limited to, one or more of the following patents.

US: 5,059,208; 5,064,433; 5,300,119; 5,578,083; 5,693,097; 5,919,231; 6,484,345; 6,776,797; 6,948,526; 7,025,784.

Foreign: AU655520; AU712617; AU727888; AU779087; EP0551198; EP0868156; EP1360016; GB2387211.

Other US and Foreign patents pending.

Blom-Singer®

voice restoration systems

*The Voice of Experience
Since 1978*



InHealth Technologies
1110 Mark Avenue
Carpinteria, CA 93013-2918 USA
(800) 477-5969 Toll Free
(888) 371-1530 Toll Free Fax

International
Tel: +01 (805) 684-9337
Fax: +01 (805) 684-8594

E-mail: info@inhealth.com
www.inhealth.com

© 2010 InHealth Technologies, A Division of Helix Medical, LLC
LT CATALOG-A